PROFIL'AIR

NOTICE D'UTILISATION **FR**
GEBRAUCHSANWEISUNG **DE**
ISTRUZIONI PER L'USO **IT**
INSTRUCTION FOR USE **EN**
GEBRUIKSAANWIJZING **NL**

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INFORMATION FOR THE PATIENT AND THEIR FAMILY

Why have you been prescribed this pressure sore prevention support? Your health condition reduces your mobility and exposes you to the risk of pressure sores.

What is a pressure sore? A pressure sore is a more or less deep skin lesion caused by excessive and prolonged compression of tissues between the body and a support surface. This excessive pressure can cut off blood circulation and lead to a sore. Pressure sores can take various forms: a simple redness lasting more than a day, skin hardening, a more or less deep wound that in severe cases can reach the muscles or underlying bone. The occurrence of a pressure sore can be linked to a lack of mobility and/or a chronic disease.

How does this support work? This support reduces pressure and allows better blood circulation in the skin and subcutaneous tissues, contributing to the prevention of pressure sores.

SPECIAL WARNINGS

CAUTION: the primary prevention of pressure sores while sitting is to lift yourself regularly and as often as possible to limit prolonged compression of the vessels over bony prominences and to allow tissue perfusion.

CAUTION: A SUPPORT ALONE IS NOT SUFFICIENT TO PREVENT PRESSURE SORES. OTHER PREVENTION MEASURES ARE ALSO ESSENTIAL: frequently change position (at least every 2 to 3 hours), maintain skin hygiene and avoid maceration, change protections regularly in case of incontinence, observe or have someone observe the skin condition daily, ensure sufficient and appropriate nutrition, drink regularly and in sufficient quantities.

IF ANY OF THESE MEASURES CANNOT BE FOLLOWED, IT IS ESSENTIAL TO INFORM YOUR DOCTOR OR NURSE AS SOON AS POSSIBLE.

CAUTION: promptly report any ABNORMAL EVENT to your doctor or nurse, such as FEVER, PAIN, or any REDNESS or BLANCHING of pressure points (ischium, coccyx, trochanter, head, shoulder, back, hip, scapula, pelvis, heel, etc.).

We inform you of the importance of a follow-up consultation with your prescriber within a month after purchasing the cushion.

CAUTION: an improperly inflated cushion is ineffective in preventing pressure sores. We recommend regularly adjusting the pressure. The cushion should only be inflated with the inflating device provided upon delivery.

CAUTION: always use the PROFILAIR cushion with its cover (non-slip surface on the seat side) to maintain the air cells and avoid direct contact with the cushion surface. Do not use with a cover other than the one provided.

The cushion contains natural latex.

- It is important to limit the thickness between the body and the support as much as possible, except for underwear and any full change. Prefer loose-fitting cotton underwear, if possible without seams in pressure areas.

- Do not interpose: folded towel or sheet, additional cushion, etc. Ensure there are no foreign objects such as: tubes, crumbs, greasy substances, etc. Limit thickness between the body and the support as much as possible.

CAUTION: For hygiene reasons, each pressure sore prevention support should be reserved for one person only.

CHECKS BEFORE FIRST USE

The box should contain: 1 pneumatic cushion, 2 covers, 1 repair kit, and 1 adjustment manometer.

INTENDED USE

Class I Medical Device - Series cushion for pressure sore prevention - Indicated for patients at high or very high risk of ischial pressure sores or with a maximum stage 1 pressure sore (presence of erythema) without stability disorder (single compartment) or with stability disorder (multi-compartment).

USAGE RESTRICTION / PRECAUTIONS FOR USE

- Check that the cushion dimensions are compatible with the chair.
- The cushion should be installed on a flat surface with the cells on top. Check that the cushion height does not impair the effectiveness of the lateral (armrests) and leg (footrests) supports.
- Ensure correct installation direction: the valve(s) should be positioned forward and to the side of the seat for accessibility.
- Before installing the patient, slightly inflate the cells by connecting the inflating device (manometer) provided with the cushion.
- Once the patient is well positioned on the cushion, adjust the cushion pressure using our color scale manometer (Conforms to NF EN 1060 standard Tolerance $\pm 3\text{mmHg}$). To distribute the air correctly throughout the cushion, proceed in stages by opening the knob several times to release air.
- For optimal use: the needle of your manometer should always be in the green zone and never exceed 55mmHg. The values of the green zone can vary between 36 and 45mmHg depending on the patient's morphology.

CAUTION: the manometer contains sensitive elements and should be handled with care - DO NOT BLOCK THE NOZZLE WITH YOUR FINGER - DO NOT APPLY TOO MUCH PRESSURE WITH THE INFLATING BULB - DO NOT DISMANTLE THE GLASS.

You can also slide your hand, palm up, between the cushion base and the patient's bony prominences. Once the hand touches the cushion base, the adjustment is correct. For better efficiency, the patient should sink well into the cushion so that there remains 1 to 2 cm of air between the bony prominences and the cushion base.

We recommend adjusting the cushion pressure regularly, not exceeding 10 days.

Acceptable patient weight: Minimum: 15 kg Maximum: 120 kg

PRODUCT DESCRIPTION

PROFILAIR is a seat cushion with individually deformable telescopic pneumatic cells.

2 versions: Single compartment or two compartments, 2 heights: 5 and 7 cm.

R-SOFT Technology = Innovative eight-terminal cell ensuring optimal ventilation while maintaining better pressure distribution. Delivered with an aesthetic cover made of breathable fabric (Polyester + PVC) and a hygienic cover made of coated fabric (Polyurethane + PVC)

CATALOG REFERENCES

REC CPA 002 - 40x40x5 cm - Single compartment 49 cells

REC CPA 003 - 40x45x5 cm - Single compartment 56 cells

REC CPA 004 - 45x45x5 cm - Single compartment 64 cells

REC CPA 005 - 40x40x7 cm - Single compartment 49 cells

REC CPA 006 - 40x45x7 cm - Single compartment 56 cells

REC CPA 007 - 45x45x7 cm - Single compartment 64 cells

REC CPA 008 - 40x40x7 cm - Two compartments 49 cells

REC CPA 009 - 40x45x7 cm - Two compartments 56 cells

REC CPA 009B - 45x40x7 cm - Two compartments 56 cells

REC CPA 010 - 45x45x7 cm - Two compartments 64 cells

REC-CPA 017 - 45x50x7 cm - Two compartments 72 cells

REC-CPA 018 - 45x50x7 cm - Single compartments 72 cells

REC-CPA 019 - 50x45x7 cm - Two compartments 72 cells

REC-CPA 020 - 50x50x7 cm - Single compartment 81 cells

REC CPA 021 - 50x50x7 cm - Two compartments 81 cells

USING THE REPAIR KIT

In case of an air leak:

- 1 Inflate the cushion and, if necessary, submerge it in water to visualize the leak (air bubbles = leak).
- 2 Clean the surface to be repaired using the alcohol wipe provided in the repair kit.
- 3 Deflate the cushion and seal the hole with the patch, let it dry for 1 hour.

CLEANING AND DISINFECTION

Wear gloves to avoid any risk of infection

• **THE CUSHION:** Submerge it in water containing an antiseptic soap. Scrub the cells at their base with a soft brush or glove. Rinse thoroughly with clear water and let dry before use or storage.

CAUTION: do not use bleach or corrosive products.

• THE COVER:

Machine washable at a maximum temperature of 40°C on a delicate cycle.

Do not use bleach - The cover can be disinfected with a detergent and disinfectant product.

Do not iron the cover.

Do not tumble dry - Air dry only.

Do not dry clean.

STORAGE / DISPOSAL

70%

+10°C

Store the cushion flat and its cover away from heat, light, and moisture. Ensure it is thoroughly dried before storage. In case of disposal, do not burn or discard in nature, follow current legislation.

WARRANTY

The cushion is guaranteed against any manufacturing defect, subject to use in accordance with the manufacturer's recommendations and proper maintenance. The warranty does not cover products used incorrectly or damaged in an accident or force majeure (fire, flood, etc.). Any modification to the product and any use of accessories not provided with the product will void the warranty.

The dealer's stamp must be affixed, and the patient's purchase date noted below, along with the serial number located on the back of the product. In case of a manufacturing defect and if the product is under warranty, please contact your dealer with this certificate and the product. The dealer will take the necessary steps with our company.

With proper maintenance and use, a PROFILAIR will last at least three years (depending on usage intensity and patient weight - except for the cover).

To find out more about our guarantee, please refer to our general terms and conditions of sale.

LEGAL INFORMATION

The general design, as well as the texts, images, know-how and any other element making up the product are the exclusive property of RECFRANCE and/or its suppliers.

Non-contractual visuals.

Any serious incident involving the device must be notified to the manufacturer and to the competent authority of the Member State in which the patient is established.

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SAS au capital de 141 300€ RCS Béziers 401 406 871 00020